



FIELDS OF APPLICATION

- Especially suited for heavy duty use
- Long-term provision with high adjustment demands for independent mobility
- Mostly implemented at home, but also in rehabilitation clinics to achieve therapeutic goals
- Implementation on different forms of drives

ADJUSTMENT

- Manifold adjustment possibilities to the body measures of the user
- Seat width from 50 – 58 cm
- Chassis adjustment by means of angle adjustable castor stem and rear wheel adapter plate
- Back rest adjustment with adjustable straps
- Seat depth adjustment from 43 - 46 cm serially
- Selection of different armrests and leg supports
- Wide selection of accessories
- Starting at 16.5 kg overall weight
- User weight up to 160 kg

OPERATION

- Foldable
- Handrim drive/ push drive
- Leg supports can be swivelled inwards or outwards
- Lockable arm supports, foldable towards the back
- All rear wheels incl. quick-release axle
- Pressure brake
- Drumbrake for attendant (option)
- Step aid serially

ACCESSORIES

- Walking stick holder
- Reflectors
- Anti-tippers
- Push bar
- Seat belt
- Brake lever extension
- Therapy table
- Auxiliary drive



Push bar

Code 814

The push bar attached between the back tubes enables stable driving behaviour and can simultaneously be used as a comfortable pushing aid.



Foot plate

Code 54

The one-piece footplate can be folded up on one side and thus also allows the easy folding of the wheelchair.



Stability

The reinforcement elements used in the rear frame ensure a stable relief of the serial back attachment.



Folding procedure

After loosening the push bar locking device the Eurochair Vario XXL can be folded without any problems.

ESPECIALLY DURABLE!



The targeted implementation of rigid frame connection elements provide the folding wheelchair with the stability required for its field of application. A double cross brace guarantees high frame stability and an excellent frame connection.



Seat cover

Also the seat cover was reinforced in the.

TECHNICAL DATA

Seat width in cm	50 / 53	53 / 56	58 / 61	65 / 68	User weight	160 / 200
Width unfolded in mm	680 / 710	710 / 740	760 / 790	830 / 860	Empty weight	16
Width folded in mm	280 / 310				Permitted overall weight	176
Seat depth in cm	43 / 46				Transport weight	10
Seat height front / rear in cm	variable 43 to 51,5				Driving wheel	600 (24")
Overall height in mm	960					
Back height in cm	42 / 44					
Arm support height in mm	230					
Length with foot support in mm	maximum 1070					
Length w/o foot support in mm	maximum 840					

Weights in kg, wheels in mm, tolerances \pm 1 cm