


Delta Plus


QUICK GLANCE USER GUIDE

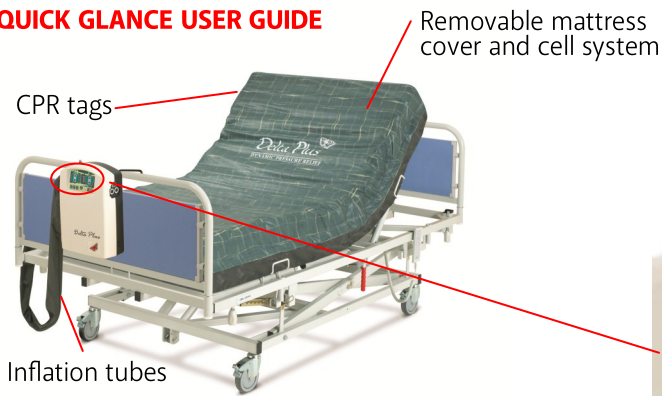

Max. Patient Weight Limit 50 stones/318 kgs

Comfort Range
Use UP/DOWN function to adjust to comfort levels (1 -softest, 6 - firmest)

Cycle Time
Use UP/DOWN function to adjust the cycle time through 10,15,20 and 25 minutes

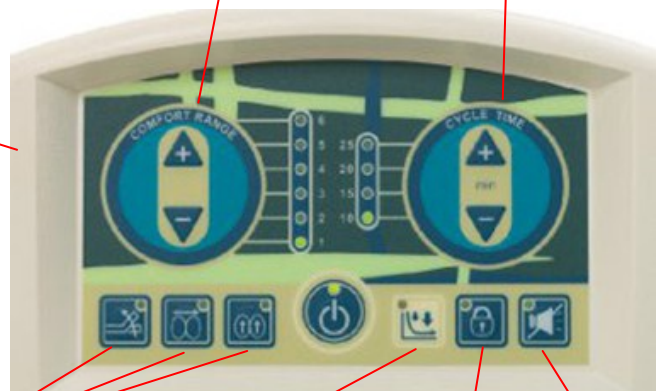

Autofirm mode

For maximum inflation of the mattress. All other functions will be disabled unless this button is pressed. This mode will operate for 20 minutes and then return to default settings automatically

Static Mode

Non-alternating mode To facilitate nursing procedures. To resume the alternating mode press this button again

Semi-recumbent mode

This will increase the internal cell pressure by 10% to accommodate the semi-recumbent position

Therapy Mode
From left to right: Semi-recumbent, Static & Autofirm

Low pressure indicator
No light - normal condition/mattress operating accordingly: Yellow - pressure has not reached the pre-set setting; Flashing Yellow - low pressure

Panel lock out
Press to lock the control panel. Press and hold for 3 seconds to unlock

Alarm Mute
Press to temporarily silence the alarms for 3 minutes



Park House
Healthcare

NEED ASSISTANCE? TEL. 0845 0600 333

Delta Plus

MODE SETTINGS & USER GUIDELINES



Low Pressure & Alarm functions

If the pump unit detects low pressure, the Low Pressure indicator (flashing yellow LED) will illuminate and the alarm will sound.

If the pressure level is constantly low, check for any leakage in the air cells or connecting tubes.

Quick set-up

1. Remove the existing mattress and place the Delta Plus system directly onto the bed frame with the cells uppermost and the inflation tubes at the foot end of the bed. Fasten the securing straps around the bed frame to secure the system.
2. Hang the pump at the foot end of the bed, adjust the hangers to secure the pump against the foot board.
3. Connect the air tube connector from air mattress to the pump unit. When a 'click' is felt/heard, the connector is secure.
4. Plug into mains electrical outlet.
5. Turn the main power switch ON. The power switch is located on the right side panel of the pump unit.
6. Press the POWER button on the display panel to start the system. For faster inflation, the Autofirm function can be activated. This mode will last for 20 minutes and then it will return to default settings automatically.
7. Once the mattress is fully inflated, the pump settings can be changed to accommodate patient comfort or if recommended by physician

CPR requirement

If rapid deflation of the mattress is required, simply pull the CPR tags or disconnect the hose at the pump to deflate the mattress within 10 seconds.

NEED ASSISTANCE? TEL. 0845 0600 333


Park House
Healthcare