

# Elite


Max. Patient Weight Limit 30 stones / 190 kgs

## QUICK GLANCE USER GUIDE



### Cushion mode

Allows the pump unit to operate the Elite Seat

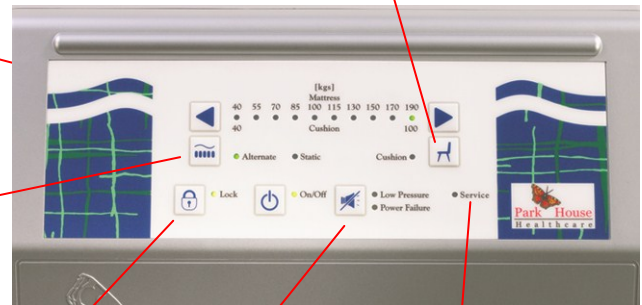

### Therapy Modes

Press this button to change from Alternating mode to Static mode and viceversa.

**Static Mode** provides a firm support surface for a healthcare professional to work on. Once selected, after 20 minutes the system will automatically revert back to Alternating mode.

**Alternating mode** provides 1 in 2 cell alternating therapy over a 10 minute cycle.

**Manual control panel lock out**  
Press to lock the control panel and prevent unauthorised setting changes

**Alarm mute**  
Press to temporarily silence the alarms

**Service indicator**  
Illuminates to indicate that the pump has detected a mechanical failure



**Park House**  
Healthcare

NEED ASSISTANCE? 0845 0600 333

# Elite

## WEIGHT SETTING & USER GUIDELINES

### Patient Weight Settings



### Quick set-up

1. Place the Elite system directly on to bed frame with the cells uppermost and the inflation tubes at the foot end of the bed.
2. Fasten the securing straps around the mattress to secure the system
3. Connect the tubes to the pump unit; plug in and turn on via the green neon switch on the right hand side of the pump.
4. Select the appropriate weight setting for the patient. The low pressure light will illuminate until the system has reached the appropriate pressure.
5. Once the appropriate pressure is reached the system will automatically start in alternating mode and the low pressure light will go out.

### Automatic panel lock out

After 5 minutes of inactivity the panel will automatically lock to prevent unauthorised setting changes. To unlock hold the panel lock button for 3 seconds.

### Low pressure / Mains failure alarms

If the pump unit detects an unusual low pressure an intermittent alarm will sound.

If the pump unit detects that the mains power has been switched off a constant alarm will sound. Pressing the ALARM MUTE button will silence the alarm for 3 minutes.

If the alarm resumes refer to the 'Troubleshooting' section of the user manual or contact Park House Healthcare.

### Cushion Mode

The pump unit can also be used with the Elite Seat

1. Disconnect the tubes from the mattress and join together to retain static pressure in the mattress.
2. Connect the tubes from the cushion to the pump unit.
3. Press and hold the cushion mode button for 3 seconds, the pump will start inflating the cushion.
4. Ensure the correct weight setting is selected. The low pressure light will illuminate until the cushion has reached the appropriate pressure.
5. Once the appropriate pressure is reached the cushion will automatically start alternating and the low pressure light will go out.

NEED ASSISTANCE? 0845 0600 333

**Park House**  
Healthcare